

**HOW TO MEASURE FRAME WRAP / ZTILT**

Place the frame, top side down, over the highlighted gray frame image. Ensure the frame is placed as it is shown in the picture. Record the angle of frame wrap from the extreme right side of where the lens ends, not where the frame ends.

**PLEASE NOTE:**

This chart will not work as a measurement tool unless printed at exactly 1 to 1 scale. Be sure that your print options are set to page scaling: none. DO NOT select "fit to printable area".

1 cm




**DEFAULT PERSONALIZATION PARAMETERS**

Wrap	Panto	Vertex
15°	12°	14 mm

**ENGRAVING INDEX**

M = Material DC = Design Code ADD = Addition Power

Product	Design Code
SportWorks Progressive	ESp

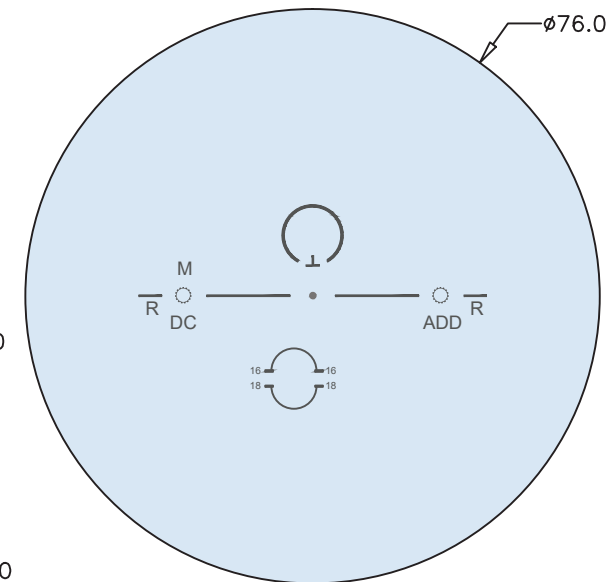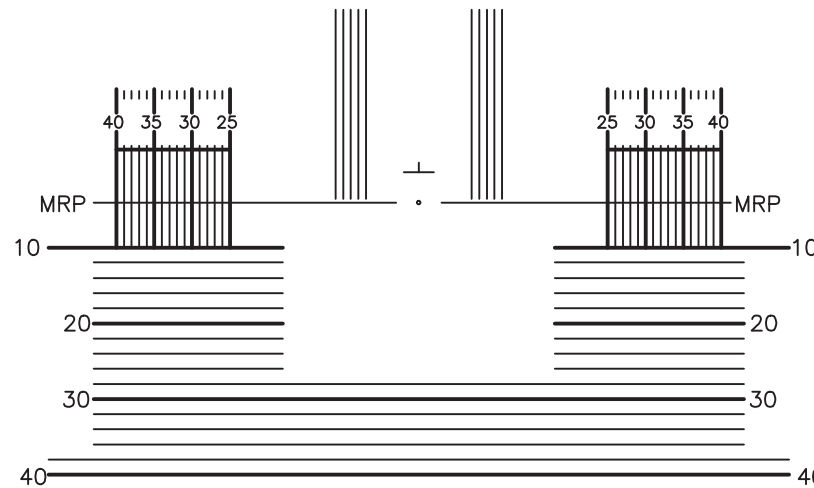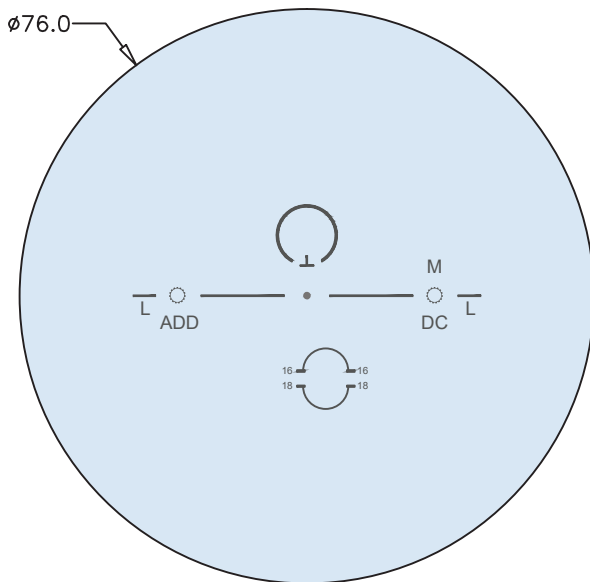

SportWorks Progressive lenses are open format designs, compatible with any spherical lens blank. Drop = 4 mm. Recommended MFHs are 16 and 18 mm. Contact your laboratory for more information.